


RT300 Series II Quick Setup Instructions¹

1. Use your *User Name* and *Password* to create a new patient record at RTILink.com.
2. Setup patient:
 - a. Adjust the pedals according to the patient's size to achieve the desired range of motion when pedaling.
 - b. Secure the patient's wheelchair using the straps that are located at the base of the machine.
3. Click "Clinician" and login.
4. Click "Patient" to open the appropriate patient.
5. Click "Therapy" to select your chosen therapy.



1. This guide created by Rob Sykes, SPT, and was adapted from setup instructions provided by Restorative Therapies.


6. "Configure" allows you to specify which electrodes are in use and adjust the stimulation parameters for each electrode.

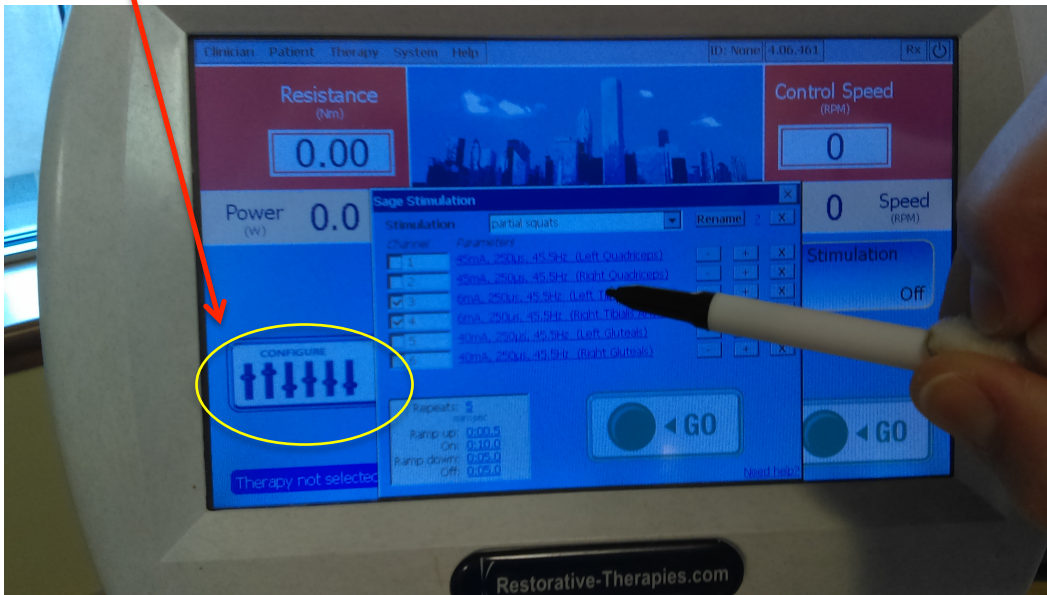

7. Adjust the parameters of each individual channel to achieve the desired response.
8. To begin, click "GO." This will begin a passive warm-up period (no stimulation). Following the passive warm-up is an active transition phase, which is then followed by active therapy. Following active therapy is a passive cool down to conclude the treatment.



9. If at any time during the treatment session the stimulation needs to be stopped, click "STOP."

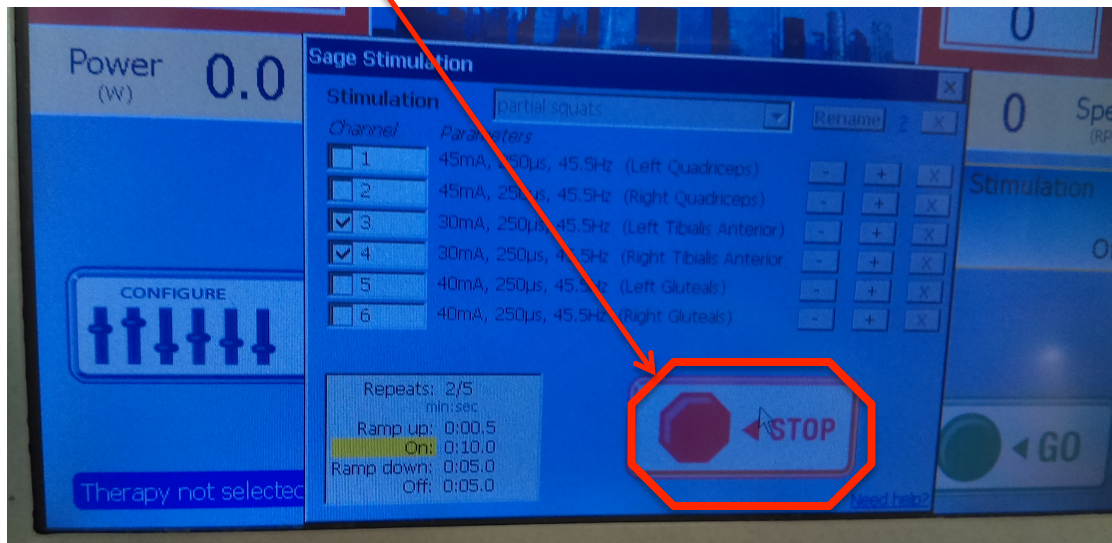

Important to Know:

- Check the patient's skin before, during, and after treatment to ensure no adverse responses to the intervention.
- Objective data (e.g., distance propelled and average asymmetry) can be recorded and saved to track patient performance and progress.